
MED SYSTEMS

Electro Flo® 5000
Airway Clearance Device

Self- Administrator
Instructions for Use



The Self-Administrator is a strap that holds the Electro Flo® 5000 Airway Clearance Device power head. With the Self-Administrator, a person can apply CPT directly on their back, unassisted, and precisely where needed for immediate relief.

The Self-Administrator can be used by anyone with manual dexterity.

Instructions

1. Place the Self-Administrator on a flat surface with the opening toward you and the Med Systems logo on top.



2. Open the Velcro closure and insert the Electro Flo® 5000 power head as shown.
 3. Tighten the Velcro closure securely.
 4. Place the Self-Administrator with the bottom cover of the power head against the back wherever needed.
 5. Adjust the straps to fit your reach, keeping the straps taut and the Electro Flo® 5000 tightly against the back. If the power head slips, pulling toward the chest will correct the position.
 6. Turn the Electro Flo® 5000 to the desired settings on the control box. Adjust as needed.
-



7. If you assume the postural drainage position, adjust the straps and your grip in order to keep the bottom cover of the power head firmly against your back.



8. When treatment is finished, remove the power head and allow it to remain in stand-by position on a lint free surface, before turning off.
 9. To clean the Self-Administrator, wipe down with soap and cool water or hand wash in cool water and air dry out of direct sunlight. To disinfect, wipe surfaces with alcohol.
-

Med Systems, Inc.

2631 Ariane Drive
San Diego, CA 92117
(858) 483-9671
(800) 345-9061

www.medsystems.com

WARRANTY

Med Systems warrants the Self-Administrator against defective materials or workmanship for the life of the product.



Made in the USA

American ingenuity

We are a family-owned company
providing respiratory care solutions since 1976
