



What People Say

"We LOVE this percussor! We've been using it intensively. It helped our daughter increase her pfts from 38 to 41 after just a few days of use. Our doctor was so happy! Moving up each point was like climbing Mt. Everest."

—Mary A.

"There are very few constants in the world of health care, with one glowing exception, Fluid Flo Percussors. Simplistic perfection for patients in need of Chest Percussion Therapy. They work well and forever, thank you for your quality device."

—John- Mendota, IL. Clinician

Warranty

The Fluid Flo® is warranted against defective material or workmanship for one year after date of purchase. The Fluid Flo® percussor is intended for use by trained personal. Med Systems shall, upon receipt of written claim of defective material or workmanship and upon inspection of parts claimed to be defective, repair, or at its option replace any part that Med System determines to be defective.

Sales and Service

(800) 345-9061

(858) 483-9671

7:00 am to 4:00 pm PST
Leave a message after hours

Fax

(858) 483- 9827

Email

sales@medsystems.com

American Ingenuity



We are a Family owned company providing respiratory care solutions since 1976. All Fluid Flo® percussors are Proudly Made in America

ISO 13485 Registered facility
Canadian Establishment Licensed
CMDCAS Licensed Registrar
ORCA/CCR Registered



SAI GLOBAL
ISO 13485
Medical
Devices



Intertek

MED SYSTEMS®

2631 Ariane Drive
San Diego, CA. 92117

www.medsystems.com

MED SYSTEMS®



Fluid Flo®
Models 2500, 2700, 2900
Handheld Pneumatic Percussors

Airway Clearance that Works!

Fluid Flo® 2500, 2700, 2900 *pneumatic percussors*



Why the Fluid Flo®?

- It delivers deep, smooth pulses that successfully clear a patient's lungs
- It is easy to use and lightweight - only 2 Pounds
- It tirelessly mimics hand percussion without fatiguing the patient or caregiver
- It's renowned for its reliability

Call us now for a 30 day trial!
US Toll Free (800) 345-9061

www.medsystems.com

Med Systems pneumatic percussors for chest physical therapy (CPT) also known as chest physiotherapy, are more effective than oscillators, vibrators or massagers to dislodge trapped pulmonary congestion. There are over 23,000 Fluid Flo® percussors in hospitals, clinics and with homecare patients for respiratory care.

The Fluid Flo® 2500 Series

The 2500 Series Includes

1. The Fluid Flo® 2500- DISS Oxygen fitting
2. The Fluid Flo®2700-DISS Medical Air fitting
3. The Fluid Flo®2900-Oilless Air Compressor fitting

Indications for Chest Physical Therapy (CPT)

- ALS
- Bronchitis
- Bronchiectasis
- Cerebral Palsy
- COPD
- Cystic Fibrosis
- Emphysema
- Guillain-Barre Syndrome
- Multiple Sclerosis
- Muscular Dystrophy
- Myasthenia Gravis
- Occupational Lung Disease
- Pneumonia or Atelectasis
- Pulmonary Edema
- And other conditions that produce retained secretions



Simple Operating Instructions

- Connect the hose assembly to a 50psi gas source with manufacturer's hose assembly- Do not attach a flow meter as it will degrade performance. 2900 model use an oil-less air compressor with minimum 2cfm (60 lpm)
- Use a soft towel/material between the percussor and the patient.
- To activate the pulse speed, turn the knob counter clockwise. Some initial gas flow is required before the pulse begins. One entire turn may be needed to start. Two turns is full speed.
- To control the pulse intensity, hold the front cover against the patient being careful not to press too firmly.
- Vary speed and intensity as needed.
- When finished with a treatment, wipe external surface with hospital grade disinfectant wipe.